Lampe stylo de poche - Pocket penlight

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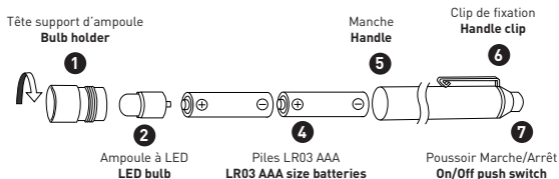
EN → This material complies with the requirements of the 93/42/EEC Medical Device Directive.

DESCRIPTION

Shock-resistant aluminium casing and handle clip.

Battery handle equipped with a LED bulb (2.8 V - 20 mA).

Accessories included: 2 LR03 AAA size alkaline batteries and 1 blade holder designed for all conventional tongue depressors.

INSTRUCTIONS FOR USE

Inserting the batteries

Unscrew the bulb holder and insert the two batteries into the handle making sure they are aligned correctly (+) and (-).

On/Off procedure

Press the on/off black push button to switch the light on intermittently. Release the button to switch the light off. To switch on the light permanently, press the upper part of the handle clip.

MAINTENANCE

Replacing the batteries

Unscrew the handle from the bulb holder, remove the low batteries and replace them with two new LR03 AAA size alkaline batteries, making sure they are aligned correctly (+) and (-).

Replacing the bulb

Unscrew the bulb holder from the handle and remove the bulb from the holder by pushing it from the outside towards the inside. Then insert the new bulb into the holder by pushing it forwards.

WARNING

- Do not soak, do not immerse into liquids.
- Avoid prolonged exposure to direct sunlight and to extreme temperatures.
- Place beyond children's reach.
- Remove batteries if the equipment is not to be used for a long time.

Operating and storage temperatures: + 10°C to + 40°C.